

PLEGA Electric Adjustable Beds

OPERATING PROCEDURES

Congratulations on your decision to purchase the Plega Adjustable Bed. Unlike regular flat beds, your Plega Adjustable Bed is a total home therapy comfort unit. Your Plega bed can be adjusted to many different positions to suit your requirements for either sleeping or relaxing. The flexible hand control unit is a simple to operate device which totally controls your bed at the touch of a button.

TO LIFT AND LOWER HEAD SECTION

You simply depress the buttons marked with upward and downward arrows. You will hear a gentle hum as the bed is being raised or lowered. You need not have fear of raising the bed too far as built in safety switches cut off the power supply when the raising or lowering has reached its limit.

TO LIFT AND LOWER FOOT SECTION

You simply depress the buttons marked with upward and downward arrows. In both cases the action will cease when your finger is removed from the button. The lift motors will not operate if your bed is hard up against a wall.

MATTRESS CARE

Our mattresses contains many layers of luxurious materials to give you a unique feeling of deep support. Slight body impressions that may appear are normal and are caused by the settling of the layers of material. In order to minimise body impressions and maximise the life of your mattress, please rotate your mattress at least once every two months.

Do not allow anyone to stand or jump on the mattress.

MASSAGE

The optional massager may make some noise when the bed is completely flat and sometimes in certain positions. Elevating the head and foot sections of the bed usually reduces the noise.

The cable tie that holds the power cord to the frame at the head of the bed should not be removed. The cable tie prevents the power cord from becoming entangled in the moving mechanism. The hand control should come out from the centre side of the bed (either the left or the right). Please make sure, that it is not entangled in the moving mechanism.

If you are using an electric blanket, please run the cords between the mattress and foundation and NOT under your bed. The cords could otherwise caught in the moving mechanism.

SAFETY DO'S AND DON'TS

DO use a Surge Protector power board to protect the electronics.

DO watch for children and small animals around your bed.

DO keep your body in the centre of the bed when changing positions.

DO NOT alter or misuse the bed or its electrical components.

DO NOT leave the bed unattended when used by an infant or invalid.

DO NOT allow liquids to come into contact with handset.

DO NOT sit or stand on either ends of the bed whilst the bed is elevated or being raised.